




UNO

One-piece Dental Implant

Creating restoration without screw channel for Cement Fixation

guide for ExoCAD software

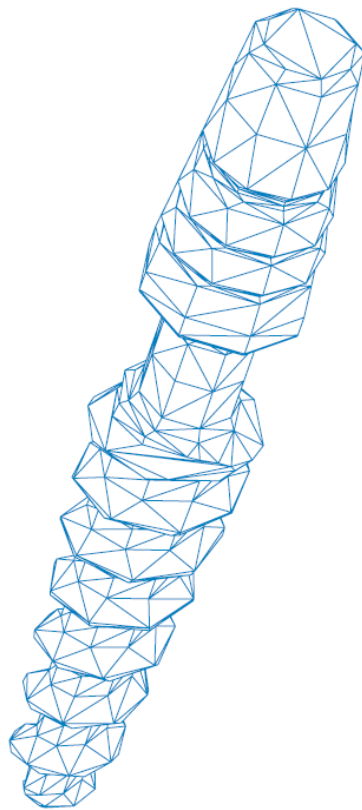

Bauer's Implants

Step 1.

Close DentalCADApp if it is open.

Step 2.

Go to: DentalCADApp/config folder on your computer.

Step 3.

Make a backup copy of defaultsettings.xml file.

Step 4.

Open defaultsettings.xml file and enter the tag line:
<ImplantCloseHoles>true</ImplantCloseHoles>
between tag lines:
<ApplicationSettingsProviderCAD>
and
</ApplicationSettingsProviderCAD>

Step 5.

Save and close defaultsettings.xml file.

Note: to turn screw channel hole feature back on, simply delete added tag line or change the value “true” to “false”.